Abbott

Clearview™

FOB ONE STEP FECAL OCCULT BLOOD TEST DEVICE (FECES)

REF

TFO-602

IVD

CE

Revision date: 2023-09-11

Number: 40003745

V01

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Package Insert

INTENDED USE

The FOB One Step Fecal Occult Blood Test Device (Feces) is a rapid chromatographic immunoassay for the qualitative detection of Human Occult Blood in feces. A rapid, one step test for the qualitative detection of Human Occult Blood in feces. For professional in vitro diagnostic use only.

SUMMARY

Many diseases can cause hidden blood in the feces. This is also known as Fecal Occult Blood (FOB), Human Occult Blood, or Human Hemoglobin. In the early stages, gastrointestinal problems such as colon cancer, ulcers, polyps, colitis, diverticulitis, and fissures may not show any visible symptoms, only occult blood. Traditional guaiac-based methods lack sensitivity and specificity, and also have diet restrictions prior to testing.^{1,2}

The FOB One Step Fecal Occult Blood Test Device (Feces) is a rapid test to qualitatively detect low levels of Fecal Occult Blood. The test uses a double antibody sandwich assay to selectively detect Fecal Occult Blood at 50 ng/mL or higher, or 6 µg/g feces. In addition, unlike guaiac assays, the accuracy of the test is not affected by the diet of the patients.

PRINCIPLE

The FOB One Step Fecal Occult Blood Test Device (Feces) is a qualitative, lateral flow immunoassay for the detection of Human Occult Blood in feces. The membrane is pre-coated with anti-hemoglobin antibody on the test line region of the test. During testing, the specimen reacts with the particle coated with anti-hemoglobin antibody. The mixture migrates upward on the membrane chromatographically by capillary action to react with anti-hemoglobin antibody on the membrane and generate a colored line. The presence of this colored line in the test line region indicates a positive result, while its absence indicates a negative result. To serve as a procedural control, a colored line will always appear in the control line region. If the control line does not appear, the test result is not valid.

REAGENTS

The test contains anti-hemoglobin antibody particles and anti-hemoglobin antibody coated on the membrane.

QUALITY CONTROL

A procedural control is included in the test. A colored line appearing in the control line region (C) is considered an internal procedural control. Control standards are not supplied with this kit; however it is recommended that positive and negative controls be tested as a good laboratory practice to confirm the test procedure and to verify proper test performance.

LIMITATIONS

- The FOB One Step Fecal Occult Blood Test Device (Feces) is for in vitro diagnostic use only.
- The FOB One Step Fecal Occult Blood Test Device (Feces) will only indicate the presence of Fecal Occult Blood, the presence of blood in feces does not necessarily indicate colorectal bleeding.
- Due to the high performance characteristic the FOB One Step Fecal Occult Blood Test Device (Feces) can obtain positive results when the specimen is below 50ng/mL in extraction buffer or 6µg/g in feces.
- As with all diagnostic tests, all results must be considered with other clinical information available to the physician.
- Other clinically available tests are required if questionable results are obtained.

EXPECTED VALUES

The FOB One Step Fecal Occult Blood Test Device (Feces) has been compared with another leading commercial rapid test. The correlation between these two systems is 97.95% (95%CI:96.92% - 98.71%).

PERFORMANCE CHARACTERISTICS

FOB One Step Test Device	Method	Other Rapid Test		Total Results
	Results	Positive	Negative	
		Positive	Negative	
	Positive	206	8	214
	Negative	14	847	861
Total Results		220	855	1,075

Relative Sensitivity: 93.6% (89.6% - 96.5%)*
Relative Accuracy: 97.95% (96.92% - 98.71%)*

Relative Specificity: 99.1% (98.2% - 99.6%)*
*95% Confidence Intervals

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PRECAUTIONS

- Each test device is for single use only.
- Standard personal hygiene measures should be taken in the case of ingestion or direct eye contact with the buffer.
- Avoid touching finger directly to the specimen pad or result window of the test as this can cause incorrect results.
- For professional in vitro diagnostic use only. Do not use after expiration date.
- The test must remain in the sealed pouch until use.
- Do not eat, drink or smoke in the area where the specimens or kits are handled.
- Do not use test if pouch is damaged.
- Handle all specimens as if they contain infectious agents. Observe established precautions against microbiological hazards throughout the testing and follow standard procedures for proper disposal of specimens.
- Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are being tested.
- The used test should be discarded according to local regulations.
- Humidity and temperature can adversely affect results.
- Do not mix buffer from different lots.
- Need to read the results in the required time.
- Proper storage condition is critical to product performance.
- Once the foil pouch is opened, use the product as soon as possible.
- Test the product in the prescribed environmental condition.
- Use the correct specimen type.
- Do not recycle the test device.
- Maybe the bystander is contaminated with the biological material and sample in the testing process, so stay away from the bystander.

STORAGE AND STABILITY

Store as packaged in the sealed pouch either at room temperature or refrigerated (2-30°C). The test is stable through the expiration date printed on the sealed pouch. The test must remain in the sealed pouch until use. **DO NOT FREEZE.** Do not use beyond the expiration date.

SPECIMEN COLLECTION AND PREPARATION

- Specimens should not be collected during or within three days of a menstrual period, or if the patient suffers from bleeding hemorrhoids or blood in the urine.
- Alcohol, aspirin and other medications taken in excess may cause gastrointestinal irritation resulting in occult bleeding. Such substances should be discontinued at least 48 hours prior to testing.

2

Sensitivity

The FOB One Step Fecal Occult Blood Test Device (Feces) can detect levels of Fecal Occult Blood as low as 50 ng/mL or 6 µg/g feces.

Specificity

Cross Reactivity

The FOB One Step Fecal Occult Blood Test Device (Feces) is specific to human hemoglobin. Specimens containing the following substances were diluted in the extraction buffer to a concentration of 1.0 mg/mL, and tested on both positive and negative controls with no effect on test results: Bovine hemoglobin, Chicken hemoglobin, Pork hemoglobin, Goat hemoglobin, Horse hemoglobin, Rabbit hemoglobin and Turkey hemoglobin.

Interfering Substance

The FOB One Step Fecal Occult Blood Test Device (Feces) has been tested for specimens containing up to 20mg/dl Ascorbic acid, 60mg/dl Oxalic acid, 100mg/dl Bilirubin, 60mg/dl Uric acid, 20mg/dl Aspirin, 2,000mg/dl Urea, 2,000mg/dl Glucose and 40mg/dl Caffeine and all the results were found negative.

Precision

Intra-Assay

Within-run precision has been determined by using 10 replicates of three specimens: a negative, a low titer positive, and a high titer positive. The negative, low titer positive, high titer positive values were correctly identified >99% of the time.

Inter-Assay

Between-run precision has been determined by 10 independent assays on the same three specimens: The negative, low titer positive, high titer positive values. Three different lots of the FOB One Step Fecal Occult Blood Test Device (Feces) have been tested using negative, low titer positive and high titer positive specimens. The specimens were correctly identified >99% of the time.

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- No dietary restrictions are necessary before using the FOB One Step Fecal Occult Blood Test Device (Feces).

MATERIALS

Materials Provided

- Test devices
 - Specimen collection tubes with extraction buffer (0.05M NaCl)
- Package insert
- Materials Required But Not Provided**
 - Specimen collection container
 - Timer

DIRECTIONS FOR USE

Allow test, specimen collection tube, specimen, and/or controls to equilibrate to room temperature (15-30°C) prior to testing.

- To collect fecal specimens:
Collect feces in a clean, dry specimen collection container. Best results will be obtained if the assay is performed within 2 hours after collection. Specimen collected may be stored for 1 days at 2-8°C if not tested within 2 hours.
***NOTE:** Considering the unexpected substances and/or microorganisms exist in feces may affect the stability of hemoglobin, it's best to test the feces as soon as possible.
- To process fecal specimens:
Unscrew the whole cap of the specimen collection tube, then randomly **stab the specimen collection stick into the fecal specimen in at least 3 different sites**. Do not scoop the fecal specimen.
Screw on and tighten the cap onto the specimen collection tube, then **shake the specimen collection tube vigorously** to mix the specimen and the extraction buffer. Specimens prepared in the specimen collection tube may be stored for 3 days at room temperature (15-30°C) if not tested within 1 hour after preparation.
- Remove the test device from the sealed pouch and use it as soon as possible. Do not touch the membrane of the strip.
- Hold the specimen collection tube upright and then unscrew and open the upper cap. Invert the specimen collection tube and **transfer 2 full drops of the extracted specimen** (approx. 90 µL) to the specimen well of the test device, then start the timer. Avoid trapping air bubbles in the specimen well. See illustration below.
- Wait for the colored line(s) to appear. **Read results at 5 minutes**. Do not interpret the result after 10 minutes.

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© 2023 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners.
Abon Biopharm (Hangzhou) Co., Ltd.
MT Promedt Consulting GmbH