

Before you begin


- Please read the instructions for use in its entirety, prior to use.
- Cover your work space with a clean, disposable absorbent workspace cover.
- All components and specimen must be brought to room temperature (between 15-30°C) prior to testing.
- Do not use the test more than once.

Preparation


- Test pouch (test device)
- Assay diluent (for Dengue IgG/IgM)
- Disposable capillary pipette (10 µl) (for Dengue IgG/IgM)
- Disposable dropper (for Dengue NS1 Ag)
- Timer & disposable gloves (not provided)

For more detailed instructions please refer to Instructions for use.



1 Check the expiration date.
If the device is expired, use another kit.

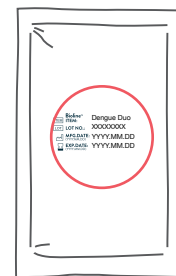


2 Put on disposable gloves.
Use new gloves for each patient.

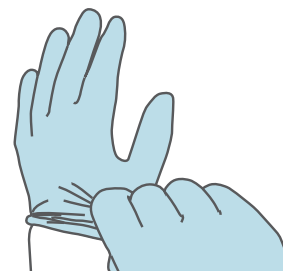


3 Open the test pouch and write the patient's name on the test device.

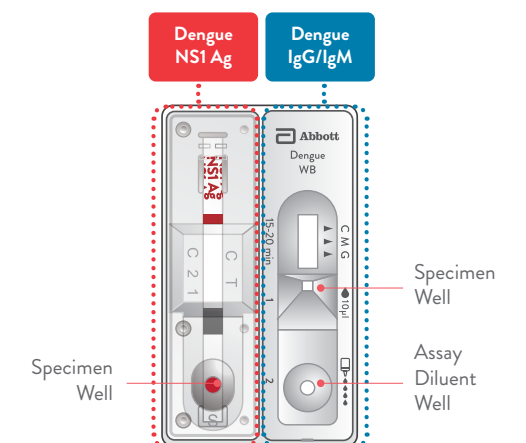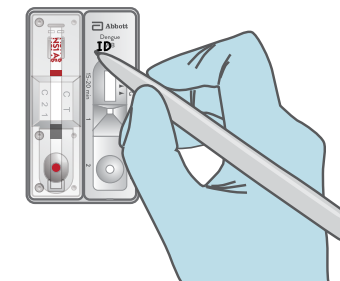


4 DENGUE NS1 Ag
Add **3 drops (100 µl)** of specimen into the specimen well ("S").
Discard the dropper in the biohazard box after use.

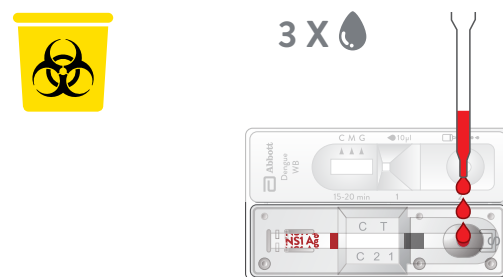

5 DENGUE IgG/IgM
Draw specimen to **black line (10 µl)** then add into the square specimen well (◀10µl).
Discard the pipette in the biohazard box after use.

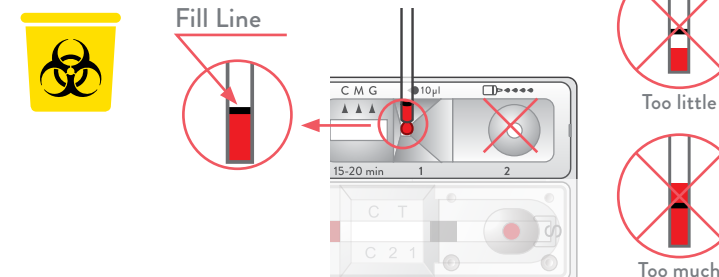

6 DENGUE IgG/IgM
Add **4 drops** of assay diluent into the round assay diluent well (◻▶▶▶▶).




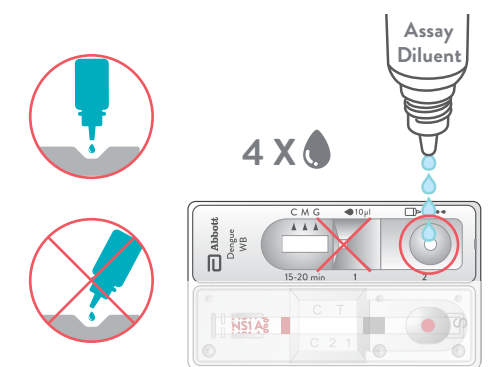

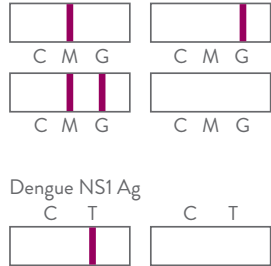


7 Read results between **15-20 minutes** after adding diluent.

Do not read after 20 minutes.
Reading outside of this time frame may provide false results.



Caution: Visible test lines should be considered positive, even if faint.

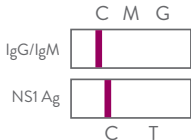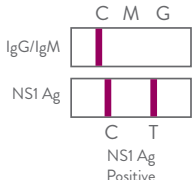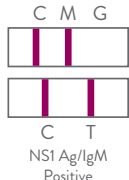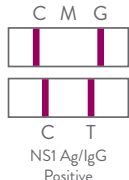NEGATIVE		POSITIVE				INVALID					
											
<p>IgG/IgM</p> <p>NS1 Ag</p> <p>C M G</p> <p>C T</p>		<p>IgG/IgM</p> <p>NS1 Ag</p> <p>C M G</p> <p>C T</p> <p>NS1 Ag Positive</p>		<p>IgG/IgM</p> <p>NS1 Ag</p> <p>C M G</p> <p>C T</p> <p>NS1 Ag/IgM Positive</p>		<p>IgG/IgM</p> <p>NS1 Ag</p> <p>C M G</p> <p>C T</p> <p>NS1 Ag/IgG Positive</p>		<p>IgG/IgM</p> <p>NS1 Ag</p> <p>C M G</p> <p>C T</p> <p>IgM Positive</p>		<p>Dengue IgG/IgM</p> <p>C M G</p> <p>C M G</p> <p>C M G</p> <p>C M G</p> <p>Dengue NS1 Ag</p> <p>C T</p> <p>C T</p>	
<p>The control line should appear for all results. If it does not appear, the result is considered invalid and the specimen should be retested using a new test kit.</p>											